Introduction

The design specifications and premium quality materials used to make the Shaker Plus ensure the device is capable of being decontaminated using various methods. The Shaker Plus has undergone laboratory testing to prove that being cleaned with enzymatic detergent alone will reduce the biological risk to an acceptable level. If greater sterilisation is required then Shaker Plus can be autoclaved to 134°C/ 273 °F at 1kg/m² pressure.

For single patient use - See the instruction in the user's manual.

For multi-patient use:

Method One - Hand Cleansing

- Disassemble the device by unscrewing the top and tipping out the ball and cone.
- Fully immerse the parts in an enzymatic cleaner such as Lancerzyme in a clean sink and wash thoroughly using either a soft cloth or cleaning pad to remove any deposits.
- Rinse thoroughly with deionised or distilled water a minimum of 3 times.
- Allow to dry naturally. Do not use a heater/hair dryer or other simulated heat source
- Reassemble and place in a clean container.

Method Two – Automatic Dishwasher

- Disassemble the device by unscrewing the top and tipping out the ball and cone.
- Place all parts in a suitable basket or on the top shelf and wash in an automatic dishwasher. The dishwasher cycle must have the following key elements
 - Pre-wash - approximately 2 mins
 - 3 minute detergents wash at 93°C / 199 °F
 - A complete rinse cycle
 - Drying stage sufficient to ensure the parts are completely dry.
- Reassemble and place in a clean container.

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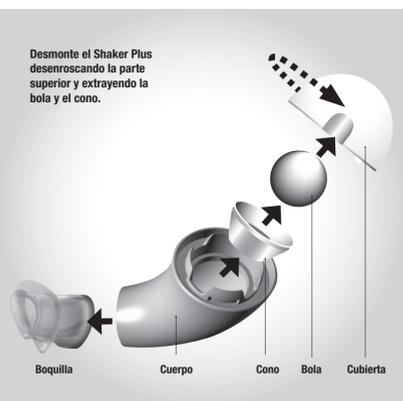
Method Three - Autoclave

After cleaning method one or method two, transfer the parts to an autoclave and sterilise at 134°C/ 273 °F at 1kg m² pressure and then place in a sterile container.

Shaker Plus: the manufacturer guarantees the decontamination (efficiency of the sterilization) of the product, without any alteration or loss of function, for up to ten (10) processes of autoclaving, except the PVC mouthpiece guaranteed up to three (3) processes of autoclaving. However, the mouthpiece coloration might change after the first autoclaving session. The discoloration itself is not a problem, but make sure that the mouthpiece functionality has not been compromised.

Note: Shaker Plus cannot be cleaned in a microwave

Cleaning and Disinfection

- Disassemble the Shaker Plus and inspect for signs of damage or wear. If any is evident replace.
- Prepare a solution of detergent in accordance with the manufacturer instructions in a container large enough to totally submerge the components.
- Leave the Shaker Plus submerged in the solution for recommended time. Wash thoroughly using either a soft cloth or cleaning pad, ensure all mucus deposits have been removed.
- Rinse as recommended. Rinse and dry as recommended.
- Pour a quantity of the chosen disinfectant into suitable container.
- Leave the Shaker Plus completely submerged in the solution for the recommended time, ensuring that there are no air bubbles.
- Rinse as recommended - Dry naturally and reassemble. (Do not use hot air or a drying cupboard).

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Disinfectants

To the best of our knowledge the following disinfectants can be used with the Shaker Plus at the stated concentrations only.

| Chemical type | Examples | Solution strength | Comments |
|-------------------------------------|---|--------------------------|--|
| Chlorine dioxide Generator | Tristel one day | 20ml in 1 litre of water | Safety data sheet and further information available from www.tristel.com |
| Sodium hypochlorite (NaOCl) | Milton | 1000 ppm | Ensure thorough rinsing, corrosion of the metal parts if exposed to chlorine for long periods. |
| Sodium dichloroisocyanurate (NaDCC) | POWERbreathre Cleaning Tablets or Preepsil, Actichlor, Sanchlor, Haz-Tab or similar | 1000 ppm | Follow instructions provided by the manufacturer. |

• POWERbreathre International Ltd accepts no liability for damage caused to products if the above procedure and recommended solutions are not used.

• It is the user's responsibility to choose which of the recommended solutions are used within their institution or hospital and we stress that the infection control nurse/department should be consulted when making the choice.

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• It is the responsibility of the user to keep them up to date with the latest information from the relevant disinfectant manufacturer concerning instructions, effects, concentrations, and immersion times.

• If your preferred cleaner/disinfectant is not on the recommended list please contact our customer service department Tel:+44 (0) 1926 816100. Email: enquires@powerbreathre.com

References: Chemical disinfection in hospitals – PHLS, Sterilization, disinfection and cleaning of medical device equipment (MAC Manual) – MHRA

Note:

- This method of cleaning has been independently tested and validated to ensure that if followed the Shaker Plus will be sufficiently clean for re-use.
- Lancerzyme® is a registered trade mark of Lancer UK Ltd (www.lancer.co.uk) other options may be available locally in other countries
- The amount of time the device is left in the solution will depend on the disinfectant chosen.
- The ball in the Shaker Plus is nickel plated carbon steel; in tests some balls have shown signs of discoloration and scratches after prolonged submersion in some cleaning solutions. However, the functionality of the product has not been affected.
- It is the responsibility of the user to ensure all sterility verifications

The Shaker Plus cannot be sterilized with ethylene oxide or hot air.

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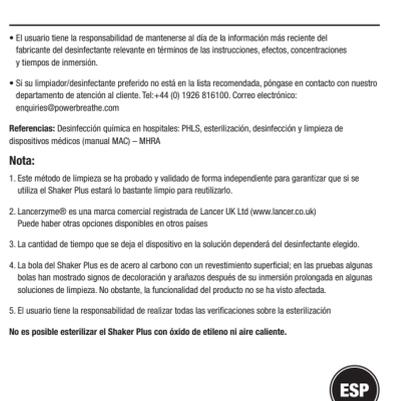
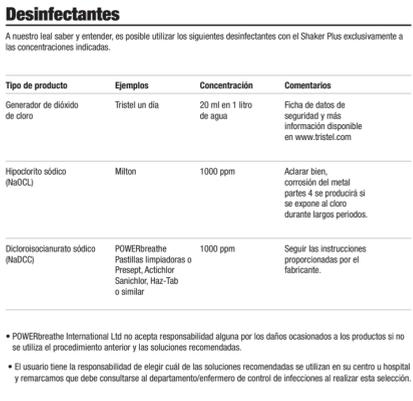
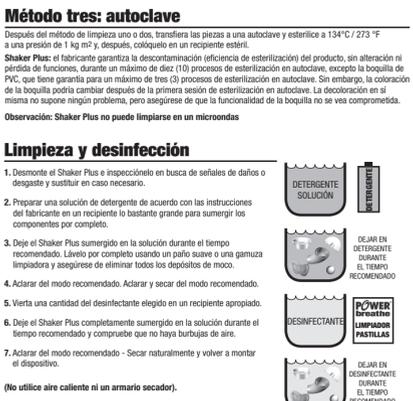
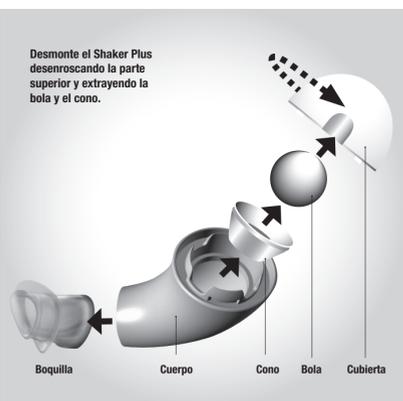
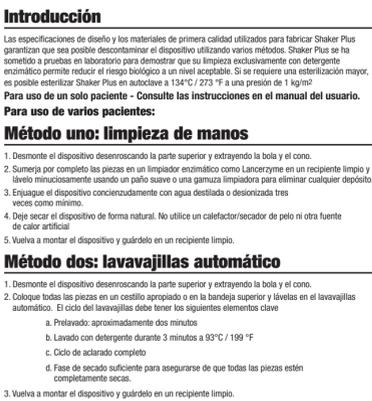
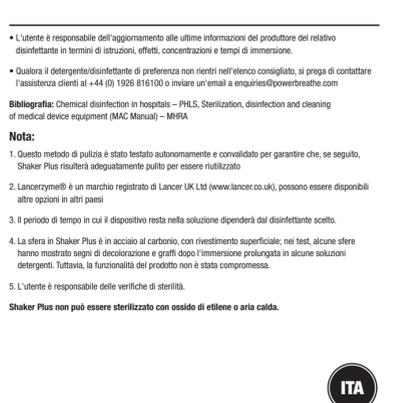
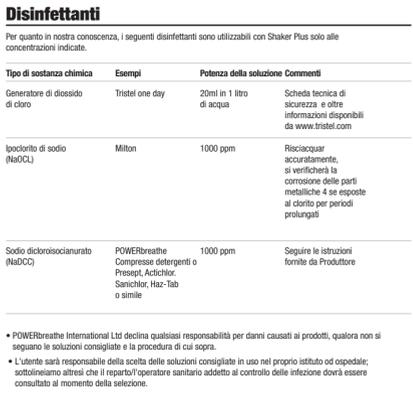
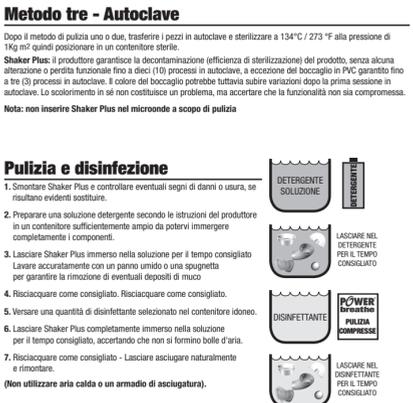
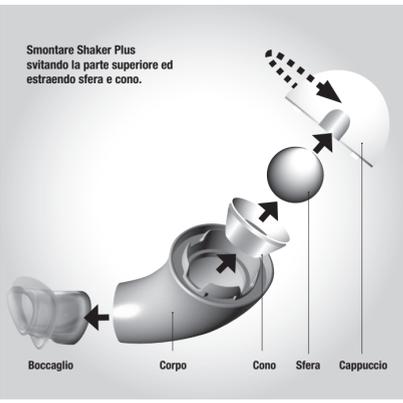
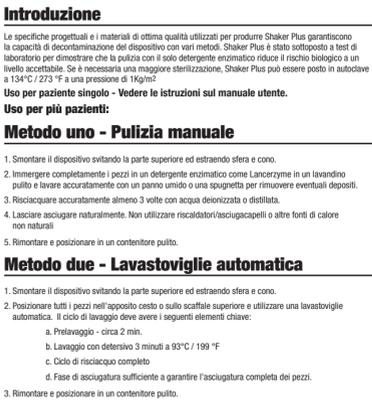
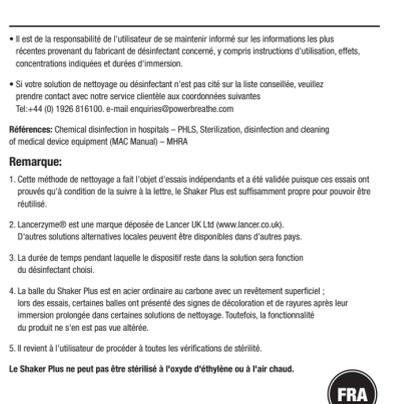
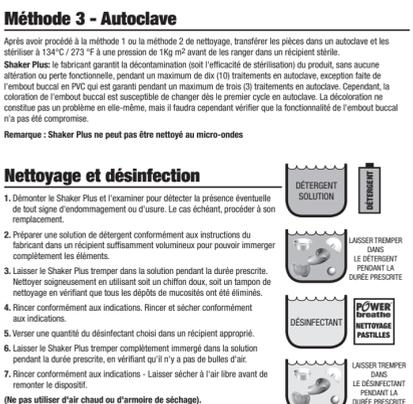
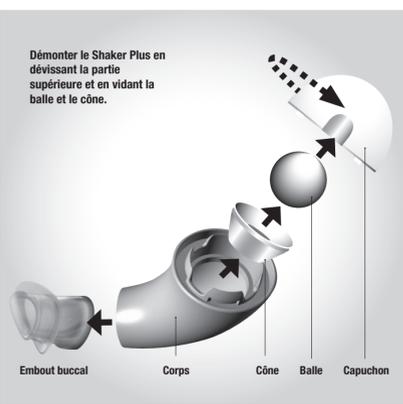
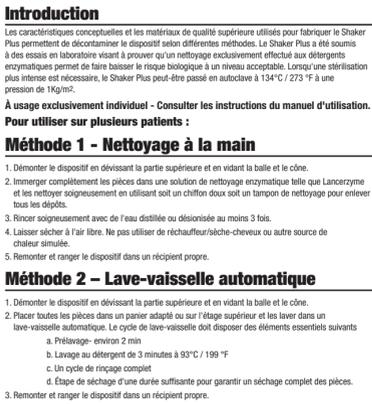
DEU

Contact information

Head Office:
POWERbreathre International Ltd
Northfield Road, Southam, Warwickshire CV47 0RD, England, UK
visit www.powerbreathre.com
Telephone: +44 (0) 1926 816100

Headline:
POWERbreathre International Ltd
Northfield Road, Southam, Warwickshire CV47 0RD, England, UK
Further Information/ Register Site visit
www.powerbreathre.com
0203 336141 or email: enquiries@powerbreathre.com
Tel: 044 (0) 1926 816100

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Made in Brazil
Exclusively for POWERbreathre® International Ltd
This product is Protected by Patent and Design Copyright ©
E & O © 01/2013.

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For Australia, New Zealand & Pacific Islands:
POWERbreathre International Ltd
Northfield Road, Southam, Warwickshire CV47 0RD, England, UK
Telephone: +44 (0) 1926 816100

Edgop:
POWERbreathre International Ltd
Northfield Road, Southam, Warwickshire CV47 0RD, England, UK
www.powerbreathre.com
Ap: 19A&9&vovvur - +44 (0) 1926 8161

總公司:
POWERbreathre International Ltd
Northfield Road, Southam, Warwickshire CV47 0RD, England, UK
欲取更多詳情, 電請請向 www.powerbreathre.com
或致我們的銷售處: +44 (0) 1926 816100

Schiedsgericht:
POWERbreathre International Ltd
Northfield Road, Southam, Warwickshire CV47 0RD, England, UK
Wwwj informacija moza zaykna na stranicj internetovj
www.powerbreathre.com
ili nazivom djelu broj: +44 (0) 1926 816100

Щедсь суджиск:
POWERbreathre International Ltd
Northfield Road, Southam, Warwickshire CV47 0RD, England, UK
Per maggior informazioni visitate il sito
www.powerbreathre.com
Tel: 044 (0) 1926 816100

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